FILLMED
LABORATOIRES

M-HA[®]
10

USE

M-HA[®]
10 | 

SYMBOLS ON THE PACKAGING

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INSTRUCTIONS FOR USE**DESCRIPTION**

M-HA[®] 10 is an injectable implant which is sterile, viscoelastic, limpid, colourless, transparent, isotonic and biodegradable.

M-HA[®] 10 is composed of sodium hyaluronate (10mg/mL) of non-animal origin, obtained by biofermentation.

3 x 3 mL vials

3 x 3 mL syringes

3 x 18G withdrawal needles

3 x 30G injection needles

3 x 32G injection needles

INDICATIONS

M-HA[®] 10 is a visco-elastic solution aimed at restoring the loss of hyaluronic acid due to skin ageing. It is designed to be injected into the upper dermis to improve the hydration, radiance and elasticity of the skin and to serve as a filling agent for small wrinkles such as crow's feet, laughter lines or lines around the mouth caused by smoking.

CONTRAINDICATIONS

M-HA[®] 10 should not be used on:

- patients prone to develop hypertrophic scars;
- patients with a history of autoimmune disease or receiving immunotherapy;
- patients with known hypersensitivity to hyaluronic acid;
- pregnant or breast-feeding women;
- patients under 18 years of age.

Patients receiving an anticoagulant treatment or a platelet aggregation inhibitor (e.g. aspirin) should not be treated with M-HA[®] 10 without having consulted their doctor beforehand. M-HA[®] 10 should not be used on areas with inflammatory and/or infectious skin lesions (e.g., acne, herpes...).

M-HA[®] 10 should not be used in combination with a laser treatment, chemical peel or dermabrasion.

PRECAUTIONS FOR USE

M-HA[®] 10 is only indicated for intra-dermal injections. Do not use if the packaging is damaged. Sensitive skin can be treated beforehand with an anaesthetic patch or cream. Please note that anaesthetic products may lead to local redness or hypersensitivity. Do not inject M-HA[®] 10 into blood vessels (by intravascular injection). Do not inject M-HA[®] 10 into bones, tendons, ligaments or muscles. Do not inject M-HA[®] 10 into moles (naevi).

Patients must be given the following recommendations: do not apply any make-up for twelve hours after the injection, avoid prolonged exposure to sunlight and ultraviolet light, do not use saunas or hammams for one week after the injection. Interaction with other agents: there are incompatibilities between sodium hyaluronate and quaternary ammonium compounds such as benzalkonium chloride solutions. This is why M-HA[®] 10 must never be put into contact with these substances or with medical or surgical instruments that have been in contact with these substances.

UNDESIRABLE EFFECTS

Before treatment the patient should be given information about the implant, its contraindications and possible undesirable effects. Undesirable effects are as follows (non exhaustive list):

- minimal bleeding at the injection site which stops quickly and spontaneously after the injection;
- transitory pain or discolouration at the injection site;
- inflammation that may be accompanied by itching or pain on pressure for one week after the injection;
- indurations or nodules at the injection site;
- erythema without swelling which disappears within one week and, in extreme cases, sometimes after two months;
- hypersensitivity to sodium hyaluronate.

DIRECTION FOR USE

M-HA[®] 10 should be injected into healthy disinfected skin with no sign of inflammation. This implant is designed to be injected into the upper dermis by a licensed medical practitioner using the appropriate technique. This treatment can be carried out manually using a 30G or 32G needle or in association with a device like an injection gun. The treatment involves micro injections into the dermal-epidermal junction or the upper dermis on the following areas:

- face
- neck
- décollete area
- backs of hands
- peri-orbital area (eye contour).

Treatment protocol consists in 3 sessions spaced out between 3 or 4 weeks.

Data relating to the injection (patient details and product lot number) should be kept by the practitioner.

WARNINGS

- Check the expiry date on the product label.
- Do not reuse.
- Do not re-sterilise.
- After use the needle should be discarded in an appropriate safety container.
- The re-use of the product bears risks (e. g. cross-contamination) for the patient.

STORAGE

Store M-HA[®] 10 at 2-25 °C (36-77 °F) in a dry place. Protect it from light, heat and frost and handle with care.

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